

## Pee-Pole II Instruction

- ①. Holding the clear cap with your left hand, & vertically with your right hand, pull the yellow tube down, and out entirely.
- ②. Still holding the clear cap in your left hand, insert the top (thicker portion) of the yellow tube into the top of the clear cap, press until it slips into place fully. Hold vertically.
- ③. Now, holding the yellow tube vertically in your hand with the clear tube pointing down, please place the clear tube with the hole in it into the urine stream. Hold in urine stream until the clear tube (bottom half) is full.
- ④ Holding the clear cap in your left hand, and keeping the device vertically, remove the yellow tube from the clear cap, invert it so the thicker portion of the tube is upright, and insert the urine filled clear tube down into the yellow tube, until it snaps fit.
- ⑤ Please don't forget writing the necessary information(name, etc)on the label and on the bag. Then, detach label at the dotted line ,peal off the silver seal and stick the label around the yellow cover. Put Pee-Pole back into the bag and seal tight.

**Please use Pee-Pole properly and only as mentioned above. Please do not use Pee-Pole for any other purpose.**