

Uro Catch II Instruction

1. Holding the clear conic container with your left hand, & vertically with your right hand, twist the yellow cover for left, and out entirely.
2. Holding the clear conic container in your hand pointing up, please place the clear tube with the hole in it into the urine stream. Hold in urine stream until the clear tube (bottom half) is full.
3. Holding the clear conic container in your left hand, and keeping the device vertically, having the yellow cover again so the thicker portion of the cover is upright, and insert the urine filled clear tube down into the yellow cover, until it snaps fit.
5. Please don't forget writing the necessary information(name, etc) on the label and on the bag. Then, detach label at the dotted line, peel off the white seal and stick the label around the yellow cover. Put UroCatch II back into the bag and seal tight.

Please use UroCatch II properly and only as mentioned above. Please do not use UroCatch II for any other purpose.

The test results of this urinalysis is to be evaluated by a doctor, as just one of other conditions, tests, results and or medical evaluations.